

13. Litvinseva, A.P., Lindsley, M.D., Gade, L., Smith, R., Chiller, T., Lyons, J.L., Thakur, K.T., Zhang, S.X., Grgurich, D.E., Kerkering, T.M., Brandt, M.E., and Park, B.J. Utility of (1-3)- β -D-glucan testing for diagnostics and monitoring response to treatment during the multistate outbreak of fungal meningitis and other infections. *J. Clin. Microbiol.* 2015; 53:618-25.
14. Olabisi, Z., Mattaruzzi, G., Estey, E., Kantarijan, H., Saeki, F., Ridge, R., Ketchum, P., Finkelman, M., Rex, J., and Ostrosky-Zeichner, L. 2004. β -Glucan as a diagnostic adjunct for invasive fungal infections: Validation, cut-off development, and performance in patients with Acute Myelogenous Leukemia and Myelodysplastic Syndrome. *CID* 39: 199-205.
15. Iwanaga, S., Miyata, T., Tokunaga, F., and Muta, T. 1992. Molecular mechanism of hemolymph clotting system in *Limulus*. *Thrombosis Res.* 68: 1-32.
16. Tanaka, S., Aketagawa, J., Takahashi, S., Tsumuraya, Y., and Hashimoto, Y. 1991. Activation of a *Limulus* coagulation factor G by (1-3)- β -D-Glucans. *Carbohydrate Res.* 218:167-174.
17. Saito, H., Yoshizaki, Y., Uehara, N., Aketagawa, J., Tanaka, S., and Shibata, Y. 1991. Relationship between conformation and biological response for (1-3)- β -D-Glucans in the activation of coagulation factor G from *Limulus* amoebocyte lysate and host-mediated antitumor activity. Demonstration of single-helix conformation as a stimulant. *Carbohydrate Res.* 217:181-190.
18. Aketagawa, J., Tanaka, S., Tamura, H., Shibata, Y., and Saito, H. 1993. Activation of *Limulus* coagulation factor G by several (1-3)- β -D-Glucans: Comparison of the potency of glucans with identical degree of polymerization but different conformations. *J. Biochem* 113:683-686.
19. Kanda, H., Kubo, K., Hamasaki, K., Kanda, Y., Nakao, A., Kitamura, T., Fujita, T., Yamamoto, K., and Mimura, T. 2001. Influence of various hemodialysis membranes on the plasma (1-3)- β -D-Glucan level. *Kidney International* 60: 319-323.
20. Kato, A., Takita, T., Furuhashi, M., Takahashi, T., Maruyama, Y., and Hishida, A. 2001. Elevation of blood (1-3)- β -D-Glucan concentrations in hemodialysis patients. *Nephron* 89:15-19.
21. Kanamori, H., Kanemitsu, K., Miyasaka, T., Ameku, K., Endo, S., Aoyagi, T., Inden, K., Hatta, M., Yamamoto, N., Kunishima, H., Yano, H., Kaku, K., Hirakat, Y., and Kaku, M. 2009. Measurement of (1-3)- β -D-Glucan derived from different gauze types. *Tohoku J. Exp. Med.* 217: 117-121.
22. Mohr, J., Patznick, V., Rodriguez, J., Finkelman, M., Coanour, C., Rex, J., and Ostrosky-Zeichner, L. 2005. A prospective pilot survey of β -glucan (BG) seropositivity and its relationship to invasive candidiasis (IC) in the surgical ICU (SICU) ICAAC Poster #M-168.
23. Held J, Wagner D. β -d-Glucan kinetics for the assessment of treatment response in *Pneumocystis jirovecii* pneumonia. *Clin Microbiol Infect.* 2011;17:1118-22.
24. Tamura, H., Arimoto, Y., Tanaka, S., Yoshida, M., Obayashi, T., and Kawai, T. 1994. Automated kinetic assay for endotoxin and (1-3)- β -D-Glucan in human blood. *Clin. Chim. Acta* 226: 109-112.
25. Ogawa, M., Hori, H., Niiguchi, S., Azuma, E., and Komada, Y. 2004. False positive plasma (1-3)- β -D-Glucan following immunoglobulin product replacement in adult bone marrow recipient. *Int. J. Hematol.* 80: 97-98.
26. Racil, Z., Koemanova, I., Lengerova, M., Weinbergerova, B., Buresova, L., Toskova, M., Winterova, J., Timikina, S., Rodriguez, I., and Mayer, J. Difficulties in using 1,3- β -D-glucan as the screening test for the early diagnosis of invasive fungal infections in patients with haematological malignancies—high frequency of false-positive results and their analysis. *J. Med. Microbiol.* 2010; 59:1016-22.
27. Posteraro, B., De Pascale, G., Tumbarello, M., Torelli, R., Pennisi, M.A., Bello, G., Maviglia, R., Fadda, G., Sanguineti, M., and Antonelli, M. 2011. Early diagnosis of candidemia in intensive care unit patients with sepsis: a prospective comparison of (1-3)- β -D-glucan assay, Candida score, and colonization index. *Crit Care* 15: R249.
28. Smith, P.B., Benjamm, D.K., Alexander, B.D., Johnson, M.D., Finkelman, M.A., and Steinbach, W.J. 2007. (1-3)- β -D-Glucan levels in pediatric patients: Preliminary data for the use of the beta-glucan test in children. *Clin. Vaccine Immunol.* 14: 924-925.
29. Goudjil, S., Kongo, G., Dasol, L., Imestouren, F., Cornu, M., Leke, A., and Chouaki, T. 2013. (1-3)- β -D-glucan levels in candidiasis infections in the critically ill neonate. *J. of Maternal-Fetal and Neonatal Med.* 26: 44-48.
30. Issa, N.C., Koo, S., Lynch, R.C., Gay, C., Hammond, S.P., Baden, L.R., Ghobrial, I.M., Finkelman, M.A., and Marty, F.M. 2012. Serum galactomannan and (1-3)- β -D-glucan assays for patients with multiple myeloma and Waldenstrom's macroglobulinemia. *J. Clin. Microbiol.* 50:1054-6.
31. Ostrosky-Zeichner, L., Alexander, B.D., Kett, D.H., Vazquez, J., Pappas, P.G., Saeki, F., Ketchum, P.A., Wingard, J., Schiff, R., Tamura, H., Finkelman, M.A., Rex, J.H. 2005. Multicenter clinical evaluation of the (1-3)- β -D-Glucan assay as an aid to diagnosis of fungal infections in humans. *Clin. Inf. Dis.* 41: 299-305.
32. Karageorgopoulos DE, Voulokmanou EK, Ntziora F, Michalopoulos A, Rafailidis PI, Falagas ME. β -D-glucan assay for the diagnosis of invasive fungal infections: a meta-analysis. *Clin Infect Dis.* 2011; 52:750-70.
33. Hou TY, Wang SH, Liang SX, Jiang WX, Liao DD, Huang DH. The Screening Performance of Serum 1,3-Beta-D-Glucan in Patients with Invasive Fungal Diseases: A Meta-Analysis of Prospective Cohort Studies. *PLoS One*. 2015 Jul 6;10:e0131602.
34. Lamoth F, Cruciani M, Mengoli C, Castagnola E, Lortholary O, Richardson M, Marchetti O. β -Glucan antigenemia assay for the diagnosis of invasive fungal infections in patients with hematological malignancies: a systematic review and meta-analysis of cohort studies from the Third European Conference on Infections in Leukemia (ECL-3). *Clin Infect Dis.* 2012; 54:633-43.
35. Onishi AI, Sugiyama D, Kogata Y, Saegusa J, Sugimoto T, Kawano S, Morinobu A, Nishimura K, Kumagai S. Diagnostic accuracy of serum 1,3- β -glucan for *Pneumocystis jirovecii* pneumonia, invasive candidiasis, and invasive aspergillosis: systematic review and meta-analysis. *J Clin Microbiol.* 2012; 50:7-15.
36. Karageorgopoulos DE, Qu JM, Korbila IP, Zhu YG, Vasileiou VA, Falagas ME. Accuracy of β -D-glucan for the diagnosis of *Pneumocystis jirovecii* pneumonia: a meta-analysis. *Clin Microbiol Infect.* 2013; 19:39-49.
37. He S1, Hang JP2, Zhang L2, Wang F2, Zhang DC3, Gong FH4. A systematic review and meta-analysis of diagnostic accuracy of serum 1,3- β -d-glucan for invasive fungal infection: Focus on cutoff levels. *J Microbiol Immunol Infect;* 2015 Aug;48:351-61.
38. Wong J, Zhang Y, Patidar A, Vilar E, Finkelman M, Farrington K. Is Endotoxemia in Stable Hemodialysis Patients an Artifact? Limitations of the *Limulus* Amoebocyte Lysate Assay and Role of (1-3)- β -D-Glucan. *PLoS One.* 2016 Oct 20;11(10):e0164978. doi: 10.1371/journal.pone.0164978. eCollection 2016.
39. Finkelman M. Specificity Influences in (1-3)- β -d-Glucan-Supported Diagnosis of Invasive Fungal Disease. *J. Fungi (Basel)* 2020 Dec 29;7(1):14

Publiccazioni di riferimento supplementari sono reperibili sul nostro sito web, [Fungitell.com](https://www.fungetell.com).

Una descrizione della guida rapida alla procedura di analisi può essere scaricata dal sito [Fungitell.com](https://www.fungetell.com):

https://www.fungetell.com/pdfs/Fungitell_ProcedureOutline_PR18-016.pdf